

**PVA Wick Drain**

**DEVICE DESCRIPTION**

Gamma sterilized PVA Wick Drain is packed in peel open pouches.

**INTENDED USE/ INDICATION FOR USE**

A sterile compressed PVA Wick Drain is used to absorb excess fluids from the corneal / scleral surface or to manipulate tissue during an ophthalmic surgical procedure.

**TARGET AGE GROUP**

Use All Age Groups

**DURATION OF USE**

Transient Use (Less than 60 Minutes).

**STORAGE CONDITIONS**

- Store at specified temperature and humidity (Temperature in between 10°C to 30°C and Humidity in between 45% to 65%).

**CONTRAINDICATIONS**

The device should not be used in patients who show hypersensitivity to PVA material.

**INDICATION/INTENDED APPLICATION**

Eyeball

**MODE OF ACTION**

In the dry state, the PVA Wick Drain is used to manipulate ophthalmic tissue. When the compressed PVA Wick Drain is brought into contact with liquid (primarily saline) it rapidly absorbs and expands to its full holding capacity. Due to this expansion, the liquid from the operating field is absorbed into the PVA Wick Drain and the operating area becomes dry.

**HOW SUPPLIED**

PVA Wick Drains are supplied sterile, in a peel open pouch. Ten (10) such pouches are enclosed within an outer printed box. The overall packing contains PVA Wick Drains and medical literature insert, in an outer enclosing box, with external identification stickers.

**MATERIALS USED**

PVA


**CAUTIONS**

Do not use the device if the sterilized package is open or damaged.

**WARNING**

- Madhu Instruments Pvt. Ltd. does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients as under: -

- The used PVA Wick Drains might not have sufficient soaking capacity and may not comply with the claim.
  - The PVA Wick Drains get deformed after wetting. The used PVA Wick Drains might not be in a position to be used again due to this deformed shape.
  - Re-use of device may lead to microbial contamination due to presence of Pseudomonas.
  - Contagious and communicable diseases such as HIV, Hepatitis, contagious disease/communicable disease/undue diseases etc.
- Only qualified medical practitioners should use the device as per the intended use.
  - Sterility is guaranteed only when the packing is neither opened nor damaged.
  - Do not re-sterilize the device by any method. Madhu Instruments Pvt. Ltd. assumes no liability for devices that have been re-sterilized.
  - Do not reuse the device.
  - Read instructions before use.
  - Store at specified temperature and humidity.
  - The used PVA Wick Drains and pouch should be disposed off in compliance with the local regulatory requirements.
  - Do not use the device after sterility expiry date.
  - For External use only.
  - Keep out of reach of children.

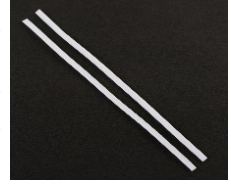

**PRECAUTIONS**

- When handling the product care should be taken to avoid damage from handling. Avoid crushing and crimping damage to the product due to application of surgical instruments.

**DIRECTION FOR USE**

- Take the sterile PVA Wick Drain pouch and peel it open (BY AN UNSTERILE PERSON) and drop the Wick Drains on to a sterile surface without touching it.
- The wick drain is gently placed on the area from which the fluid is to be absorbed.
- The sponge swells up on absorbing the fluid and ceases to absorb any more fluid after a point.
- In case the PVA wick drain is to be used for corneal tissue manipulation, dip the tip of the sponge in saline solution for few seconds. This will make the PVA wick drain very soft and suitable for corneal tissue manipulation.

**DEVICE LIFE**

The expiration date of the product is the sterility expiration date. The product should not be used after the indicated sterility expiration date.

**RETURN OF DAMAGED PRODUCT**

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor's office regarding product return/exchange.

**REPORTING**

Adverse events and/or potentially sight-threatening complications that may

reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be reported to Madhu Instruments Pvt. Ltd.

**DISPOSAL OF THE USED DEVICE**

The used device should be disposed off in compliance with the local regulatory requirements.

**EXPLANATION OF SYMBOLS:**

Symbol	Meaning
	Caution
	Manufacturer
	Date and Country of Manufacture
	Consult instructions for use
	Keep away from sunlight
	Medical Device
	Do not use hook
	This side up
	Temperature Limit
	Humidity Limitation
	Keep Out of reach of children
	Handle with care

Symbol	Meaning
	Maximum stacking
	Batch Code
	Use by date
	Do not use if package is damaged
	Do not re-use
	Single Sterile Barrier System, sterilized using irradiation
	Single Sterile Barrier System with protective packaging outside, Sterilized using irradiation
	Do not re-sterilize
	Catalogue Number
	Unique device identifier
	Model Number

**MADHU INSTRUMENTS PVT. LTD.**

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.  
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India  
 Customer Care No.: 011-42701030, 42701031  
 Email: customercare@madhuinstruments.com  
 Website: www.madhuinstruments.com