

Occluder (Multi Purpose)

Unique design of the occluder allows for eye examination by either multiple pinholes or a single pinhole. The shutter attachments can be used to cover both the eyes.



One eye occluded – one eye Single Pinhole



One eye occluded – one eye Multiple Pinholes



Both Eyes Occluded

Product Description

| Product Code | Description | Qty. per box |
|--------------|--------------------------|--------------|
| MIPL/N10/MP | Occluder (Multi Purpose) | 1 Pc. |

Features

- 1.20 mm diameter pinholes.
- Made of ABS Plastic and Stainless Steel rivets.
- Useful in numerous Ophthalmic and Optometric diagnosis.
- Ergonomic grip.
- Branding options on the handle.

Indications

- Ophthalmic and Optometric diagnosis such as detecting presence of refractive errors or testing visual acuity in mydriatic patients



Instructions for Use

- Hold the Occluder with the handle.
- Decide which function to use (1 eye Occluded - 1 eye Multiple pinholes, 1 eye Occluded - 1 eye Single pinhole or Both eyes Occluded)
- Slide the selected shutter downwards to expose the pinhole(s).
- Position the Occluder on the face to occlude the eye appropriately.

Disinfection

- Products are advised to be disinfected by 70% IPA before every use.



MADHU INSTRUMENTS PVT LTD

A-260, Okhla Industrial Area, Phase - 1, New Delhi - 110020, India

T: (+91-11) 4270 1030, 31 | E: sales@madhuinstruments.com | www.madhuinstruments.com
Customer Care No. +91 9810866176 (for international queries), +91 7428186111 (for domestic queries)

Ref. No.: MIPL/Pub/SPB/N10/MP, Issue No.: 01, Issue Date: 17.12.2021, Rev. No. 02, Rev. Date: 01.01.2023